



Patient's choice[®]
CREATED WITH PATIENTS IN MIND... from **CROSSTEX**[®]

TWIST[®] PROPHY ANGLE
First and **ONLY** disposable 90°
reciprocating prophy angle that
eliminates spatter and heat.



Features:

- Unique, patented **OSCILLATING MOTION**
- **Eliminates spatter** of prophy paste, blood, saliva and other potential infectious matter
- **Eliminates frictional heat** – remains cool for increased patient comfort
- Allows you to maintain **continuous contact and pressure** on tooth surface for maximum stain removal
- Ergonomically superior – No more “lift and dab” technique
- Safe for use on gingival margins and exposed dentin
- Ideal for applying dental medicaments, and preparing teeth for etching and bleaching procedures
- Only effective prophy angle brush for orthodontic cases – use on and in between brackets, wires and bands

Ordering Information:

100/Bag

TPASC	SOFT CUP
TPAFC	FIRM CUP
TPATB	TAPERED BRUSH
TPAFB	FLAT BRUSH





Twist is a unique disposable prophylaxis angle that moves in a 90° reciprocating back and forth motion instead of a traditional spinning motion. This patented design truly eliminates the spatter of prophylaxis paste, saliva, blood and other potential infectious matter as well as frictional heat.

INSTRUCTIONS FOR USE

1. Attach the Twist prophylaxis angle to your handpiece and run it at 4000 RPM's.
2. Maintain constant speed and contact, gliding from tooth to tooth, for as long as necessary to polish and clean all surfaces, including deep and stubborn stains. There is no need to lift, dab or dot Twist on and off the teeth. Twist will not generate any uncomfortable heat and will not stall while using firm pressure.
3. Always seat Twist completely and operate with the Twist drive shaft in alignment to the axis of the handpiece. **DO NOT** flex Twist at an angle to the handpiece.

TWIST... MORE THAN JUST A PROPHYLAXIS ANGLE.

Twist has many other uses because of its unique features and characteristics:

- **Great on Exposed Dentin** - thorough cleansing of prepared teeth prior to bonding is essential for enhanced bond strength. Twist is the only prophylaxis angle that can be used safely on exposed dentin and other prepared surfaces without producing heat and related discomfort or gingival bleeding.
- **Excellent Application/Preparation Tool** - use Twist for applying dental medicaments as well as preparing teeth for acid etchant and bleaching procedures. With Twist, you'll have excellent control resulting in exact placement... no contamination and no messy spatter. Leading clinicians have determined that rotating brushes on newly placed composites can damage the restoration. A Twist brush will NOT cause damage because of the 90° reciprocating back and forth motion.
- **Orthodontic Cases** - Twist is the only effective prophylaxis brush for use on orthodontic cases. Twist always follows the path of least resistance. This means that you can safely use it on and in between brackets, wires and bands.
- **Effective for Subgingival and/or Interproximal Use** - use of traditional rotating prophylaxis angles under the gums can result in gingival lacerations and/or bleeding. Twist's gentle, but firm back and forth motion helps to avoid gingival trauma and will not alter margins.



Hauppauge, NY 11788-4209 USA

Tel: 631-582-6777 • 888-CROSSTEX • 877-PROPHYS

Fax: 631-582-1726 • E-mail: crosstex@crosstex.com

California • Georgia • New York • Pennsylvania • Argentina • Japan • The Netherlands



www.crosstex.com