

INSTRUCTIONS FOR USE

Imaging Comfort




BiteWing-Ease® X-Ray Comfort Barriers


How to use your Crosstex® BiteWing-Ease®:



Step 1:
Insert sensor image-side up
in BiteWing-Ease.

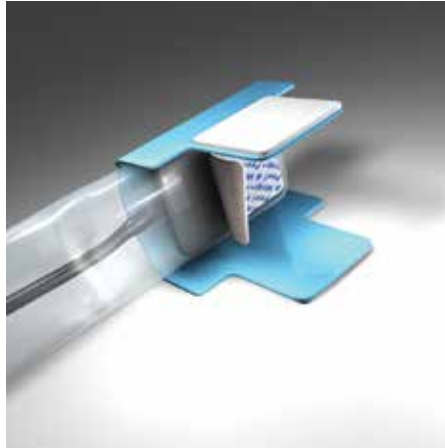

Step 2:
Remove "Peel & Wrap"
liner and fold to center
to create bite tab.

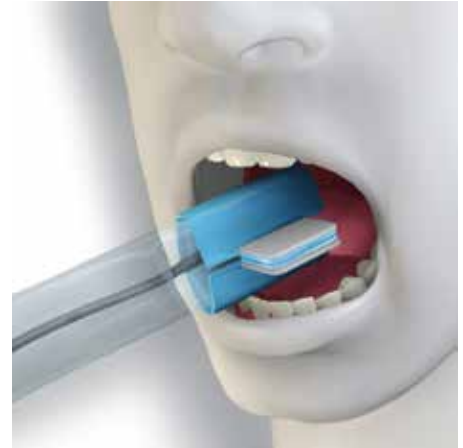

Step 3:
Take images and
discard BiteWing-Ease
after use.

Ref.#'s BWEL, BWES

BiteWing-Ease® is a registered trademark of Crosstex International, Inc.



Not made with natural rubber latex.



CROSSTEX
A CANTEL MEDICAL COMPANY

Manufactured for: Crosstex International, Inc.

10 Ranick Road | Hauppauge, NY 11788 | 631.582.6777 | 888.276.7783 | Crosstex@crosstex.com

Made in China

BWE-IFU
Rev. 0 - 11/16