

# SANI Glut™

Glutaraldehyde 3%

## When safety counts and minutes matter.

**SANI Glut** Glutaraldehyde (3%) solution with activator is a reusable sterilizing and high level disinfection solution. **SANI Glut™** is noncorrosive and is compatible for use on endoscopes and other heat sensitive surgical instruments and medical devices. Once activated, **SANI Glut** can be used up to 28 days in manual applications. **SANI Glut** provides high level disinfection in 25 minutes and sterilization in 10 hours.



spsmedical.com | 800.722.1529

**CROSSTEX**®  
A CANTEL MEDICAL COMPANY

**SPS**  
medical

# The *SANI Glut*<sup>TM</sup> Advantage

## FAST ACTING AND EFFECTIVE

*SANI Glut*<sup>TM</sup> Glutaraldehyde is fast acting and useful against a variety of microorganisms. It is readily biodegradable, formaldehyde free and performs more effectively than products that use formaldehyde, glyoxal or other corrosive materials.

## FEATURES & BENEFITS

- **EFFECTIVE** – achieves high-level disinfection in 25 minutes and sterilization in 10 hours.
- **CONVENIENT** – comes ready to use. No dilution needed.
- **ECONOMICAL** – reusable for up to 28 days\*.
- **EXCELLENT MATERIALS COMPATIBILITY** – noncorrosive and compatible for use on endoscopes and other heat sensitive surgical instruments and medical devices.
- **EASY ACTIVATION** – activate *SANI Glut* by adding the entire contents of the activator vial which is attached to the *SANI Glut* container.

## TECHNICAL INFORMATION

- In-Use Concentration: 3% Glutaraldehyde
- Soak Time High-level Disinfection: 25 minutes at 25°C.
- Sterilization: 10 hours at 25°C.
- Use Life: Up to 28 days\* once activated.
- Use Conditions: Designed for manual use.
- Shelf Life: 2 years.
- Open Bottle Shelf Life: 28 days\* (when open and activated) maximum.
- Storage: 15-25°C.
- Disposal: Drain as per facility policy. Flush thoroughly with water.

\* Solution needs to be tested with the appropriate test strip prior to each use to verify effective disinfection concentration.

## BY THE NUMBERS

**28** DAY reuse period\*

**25** MINUTE minimum high-level disinfection time at 25°C

**10** HOUR sterilization at 25°C



## Ordering Information

Description	Quantity	Ref.#
<i>SANI Glut</i> <sup>TM</sup>	1-gallon bottle,	JGLUT
Glutaraldehyde 3% with activator	4 gallons/cs	
SANI HLD Holder	Holds 1 gallon of liquid	JHOLD

**CROSSTEX**<sup>®</sup>  
A CANTEL MEDICAL COMPANY

10 Ranick Rd.  
Hauppauge, NY 11788-4209  
Tel: 631-582-6777  
[www.crosstex.com](http://www.crosstex.com)

**SPS**  
medical

6789 West Henrietta Rd.  
Rush, NY 14543  
Tel: 585-359-0130  
[www.spsmedical.com](http://www.spsmedical.com)

Manufactured for Crosstex International/SPSmedical.