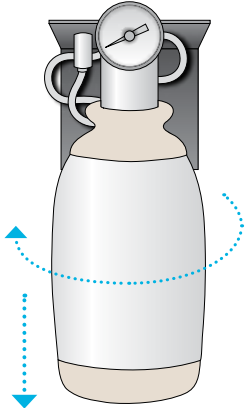


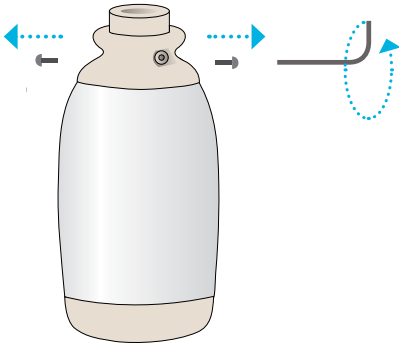
# Installing a DentaPure™ DP365B Cartridge onto an A-dec® Unit

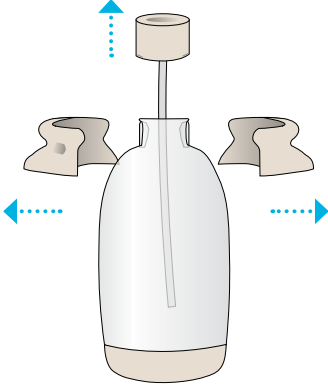


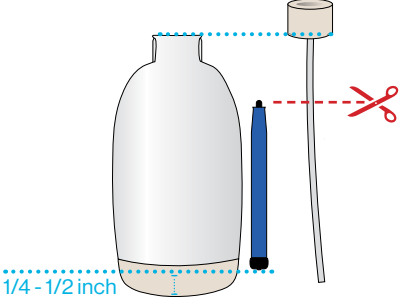
## 0.7 Liter or 2 Liter

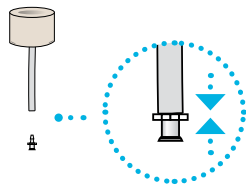
- 1** Remove the A-dec® bottle from the manifold

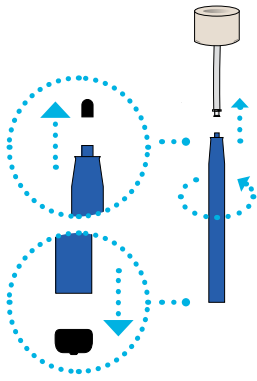

- 2** Remove two hex screws with 1/8th inch allen wrench

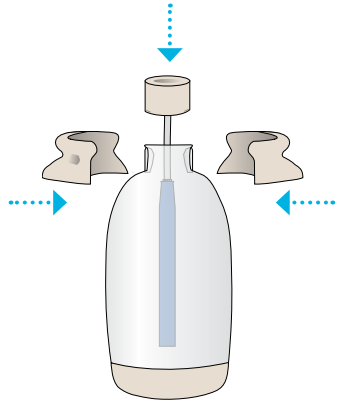

- 3** Separate collar and lift pick-up tube assembly from the bottle


- 4** Align the bottle alongside the pick-up tube to ensure that your DP365B cartridge can attach to the pick-up tube with enough space to fit inside the bottle with 1/4 - 1/2 inch of empty space from the bottom


- 5** Attach the luer fitting onto the pick-up line.


- 6** Remove the top and bottom dust caps and attach the cartridge onto the luer fitting.


- 7** Place the pick up tube assembly with the cartridge back into the bottle. Reattach the collar to the bottle, replace the hex screws, and replace the A-dec® bottle to the manifold



Learn More at [HuFriedyGroup.com/DentaPure](https://www.HuFriedyGroup.com/DentaPure)



Hu-Friedy Mfg. Co., LLC, 1666 E. Touhy Ave., Des Plaines, IL 60018 | [HuFriedyGroup.com](https://www.HuFriedyGroup.com)  
All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC its affiliates or related companies, unless otherwise noted. A-dec® is a registered trademark of A-dec, Inc.  
©2022 Hu-Friedy Mfg. Co., LLC. All rights reserved. DLIT01532 Rev A 1022

Crosstex is now a proud member of

**HuFriedyGroup**  
The Best In Practice