

Installing a DentaPure™ DP40B Cartridge onto an A-dec® Unit



0.7 Liter or 2 Liter

- 1**

Remove the A-dec® bottle from the manifold
- 2**

Remove two hex screws with 1/8th inch allen wrench
- 3**

Separate collar and lift pick-up tube assembly from the bottle
- 4**

Align the bottle alongside the pick-up tube to ensure that your DP40B cartridge can attach to the pick-up tube with enough space to fit inside the bottle with 1/4 - 1/2 inch of empty space from the bottom
- 5**

Attach the luer fitting onto the pick-up line
- 6**

Twist the luer end of the cartridge onto the cut end of the pick up tube
- 7**

Place the pick up tube assembly with the cartridge back into the bottle. Reattach the collar to the bottle, replace the hex screws, and replace the A-dec® bottle to the manifold

Learn More at [HuFriedyGroup.com/DentaPure](https://www.HuFriedyGroup.com/DentaPure)



Hu-Friedy Mfg. Co., LLC, 1666 E. Touhy Ave., Des Plaines, IL 60018 | [HuFriedyGroup.com](https://www.HuFriedyGroup.com)
All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC its affiliates or related companies, unless otherwise noted. A-dec® is a registered trademark of A-dec, Inc.
©2022 Hu-Friedy Mfg. Co., LLC. All rights reserved. DLIT01533 Rev A 1022

Crosstex is now a proud member of

HuFriedyGroup
The Best In Practice