


# CHEMICAL INDICATOR REFERENCE CHART

## STEAM

### Unprocessed



### Passed Example

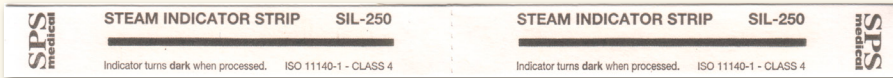

### Typical Failed Example



#### Special Note:


Accept only when indicator turns Dark in steam. Other color variations may be indicative of suspect sterilization conditions.

## STEAM FORMALDEHYDE


### Unprocessed



### Passed Example

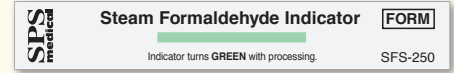

### Typical Failed Example

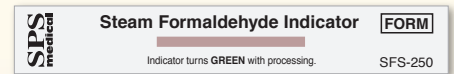

#### Special Note:

Accept only when indicator turns Green in steam formaldehyde. Other color variations may be indicative of suspect sterilization conditions.

## EO GAS

### Unprocessed



### Passed Example



### Typical Failed Example




#### Special Note:


Accept only when indicator turns Brown/Gold in EO gas. Other color variations may be indicative of suspect sterilization conditions.

## VH<sub>2</sub>O<sub>2</sub>


### Unprocessed

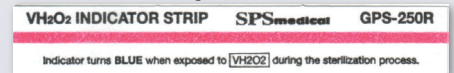

### Passed Example

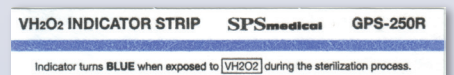

### Typical Failed Example

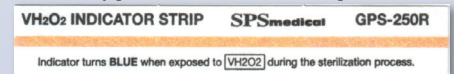

#### Special Note:

Accept only when indicator turns Blue when processed. Other color variations may be indicative of suspect sterilization conditions.