


Lead-free Steam Indicator Tape

Available in multiple widths for all your steam sterilization needs.

The process Lead-free Indicator Tape consists of an adhesive, backing and chemical indicator lines. The resilient, pressure-sensitive adhesive is designed for use with disposable wraps. After the steam sterilization process, the tape helps to keep wrapped items secured and can be easily removed.

The backing is a natural or blue-colored saturated crepe paper and provides the elongation needed for pack expansion during sterilization. The chemical indicator lines will show a visual color change from beige to dark brown/black when proper levels of moisture and temperature have been achieved. The Lead-free Steam Indicator Tape is compliant to ISO 11140-1:2014 Type 1.

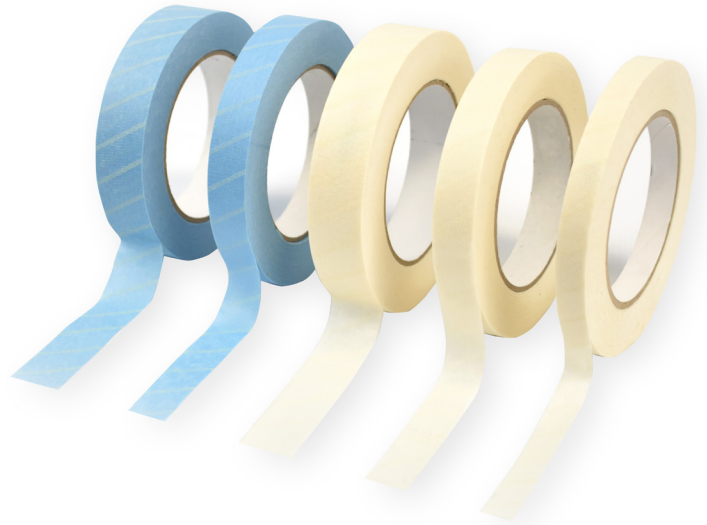

Features and Benefits

- Made with adhesive and ink that is free of natural rubber latex, lead or solvents; safer for your patients, staff and the environment.
- Validated for use with all common dynamic-air-removal (pre-vacuum/SFPP) and gravity steam sterilization cycles, allowing simplified stocking of one tape for all steam sterilization needs.
- Made with a resilient pressure-sensitive adhesive, which securely seals sterilization packs, ensuring that sterile integrity is maintained during sterilization, transport and storage.
- Distinct ink color change from beige to dark brown/black when exposed to a steam sterilization process, making it easy to interpret indication of processed and unprocessed packs to avoid mix-ups.
- Lot number and expiration date printed on the core and UDI label, allows for easy traceability.
- Saturated crepe paper provides the elongation needed for pack expansion during sterilization.
- Tape surface can be written on with indelible non-toxic ink to conveniently document appropriate information (e.g. date, sterilizer, contents, load/cycle).
- Satisfies the requirement to have a process indicator visible on the external sterilization packaging.
- The tapes (up to two rolls of 1 in tape) can be held and dispensed using the Crosstex Tape Dispenser (Ref # TD-001).

Chemical Indicator Tape

Unprocessed and Processed Tapes

Tape reaction to proper sterilization:

Natural




Before



After

Blue

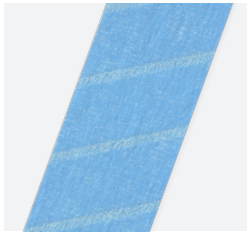

Before



After

Product ordering information:

Lead-free Steam Indicator Tape, Natural

Ref. #	Description	Quantity
STLF24MM	1 in x 60 yd (24 mm x 55 m)	18 Rolls/Case
STLF18MM	0.75 in x 60 yd (18 mm x 55 m)	24 Rolls/Case
STLF12MM	0.5 in x 60 yd (12 mm x 55 m)	36 Rolls/Case

Lead-free Steam Indicator Tape, Blue

Ref. #	Description	Quantity
BTLF24MM	1 in x 60 yd (24 mm x 55 m)	18 Rolls/Case
BTLF18MM	0.75 in x 60 yd (18 mm x 55 m)	24 Rolls/Case

Indicator Tape Accessories


Ref. #	Description	Quantity
TD-001	Tape Dispenser	Each

Call your preferred distributor to order.

Learn more at:



Crosstex.com

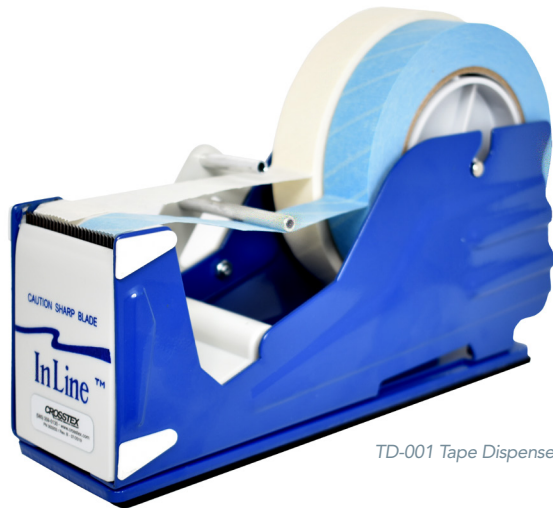

TD-001 Tape Dispenser

All product names are trademarks of Crosstex International, Inc., a Cantel Medical Company, its affiliates or related companies, unless otherwise noted.

© 2019 Crosstex International, Inc. DLIT01288 Rev A 1119