

### SporView<sup>®</sup>Plus Steam BI Test Pack

FDA cleared for use in both gravity and prevacuum steam sterilizers, the SporView<sup>®</sup>Plus Steam BI Test Pack (PCD) is equivalent in performance to an AAMI BI PCD.

#### Description

The SporView<sup>®</sup>Plus Steam BI Test Pack allows users to immediately release processed loads with peace of mind from the STEAMPlus<sup>™</sup> Sterilization Integrator, which is permanently attached to a load record card. This test pack complies with standards for routine sterilizer efficacy monitoring and biologically testing implant loads. Special Note: Media growth promotion has been validated for growth of G. stearothermophilus after exposure to a steam cycle of 132°C for 10 minutes exposure.

#### Frequency of Use

Per AAMI standards, steam sterilizer cycles should be monitored at least weekly, preferably every day the sterilizer is in use and every load that contains an implant. In addition, BIs should be used during initial installation of sterilizers, after relocation, malfunction, process failures, major repairs or for periodic quality assurance testing of representative samples of actual products being sterilized.

#### Features & Benefits

- PCD contains STEAMPlus<sup>™</sup> Sterilization Integrator allowing for immediate release of non-implant loads
- Meets AAMI recommendations for implant loads (BI and Class 5 within a PCD)
- Media growth promotion is validated per USP
- Incubators have built-in crusher




#### Reference

ANSI/AAMI ST79. Comprehensive guide to steam sterilization and sterility assurance in health care facilities.

### ORDERING INFORMATION

| Cat. No. | Description   | Packaging |
|----------|---|-----------|
| SBT-025  | SporView <sup>®</sup> Plus Steam BI Test Pack (25 controls) | 25/cs     |
| SBT-255  | SporView <sup>®</sup> Plus Steam BI Test Pack (5 controls)  | 25/cs     |
| NDB-060  | 55°C-60°C Dry Block Incubator, 11mm                         | ea        |
| SPS-100  | Steam Record Notebook                                       | ea        |

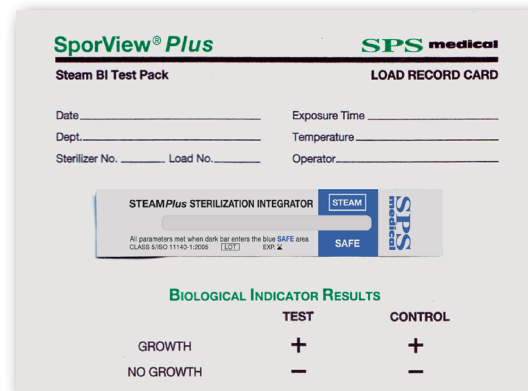

Actual size is 4.5" x 6"

**Unprocessed**

STEAMPlus and SporView are trademarks of SPSmedical Supply Corp.