

Crosstex...

Two New Spins on Propphy Angles.


Sparkle[™]
A New Angle on Propphy's

Sparkle is a 360° improvement over other rotating prophy angles. After consulting with hundreds of hygienists about what they liked and disliked about most prophy angles, we created Sparkle – a smoother-running prophy angle with a brighter, more visible body and higher contrast soft cups.

Features:

Patented 2nd Generation Beveled Gear Design For Superior Performance

- Smooth-Running
- No Over-Heating
- Vibration-Free
- Non-Locking
- No Noise

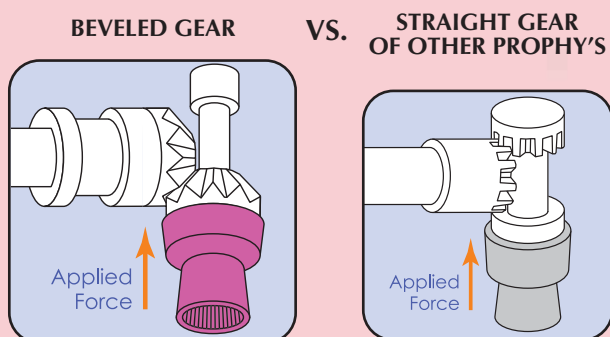

Improved Cup

- Dark Colored Soft Cups - Better visibility and contrast with teeth
- Perfect Cup Flare – Improves subgingival access
- Reduced Splatter – Helps minimize cross contamination
- 100% Latex-Free

A More Ergonomic Design

- Non-Slip Contour - Reduces hand fatigue and micro-traumas
- Light Colored Shell - For improved visibility in the mouth

Ordering Information:

Box of 100

TPASSC1	Soft, Purple Cup
TPASFC1	Firm, White Cup
TPATBC1	Tapered Brush

Box of 500

TPASSC5	Soft, Purple Cup
TPASFC5	Firm, White Cup

Patient's Choice[®]
CREATED WITH PATIENTS IN MIND... from **CROSSTEX**[®]





Lose the mess. Keep your cool.

TWIST® PROPHY ANGLE
is the first and **ONLY** disposable
90° reciprocating prophy angle
that eliminates spatter and heat.



Features:

- Unique, patented **OSCILLATING MOTION**
- **Eliminates spatter** of prophy paste, blood, saliva and other potential infectious matter
- **Eliminates frictional heat** – remains cool for increased patient comfort
- Allows you to maintain **continuous contact and pressure** on tooth surface for maximum stain removal
- Ergonomically superior – No more “lift and dab” technique
- Safe for use on gingival margins and exposed dentin
- Ideal for applying dental medicaments, and preparing teeth for etching and bleaching procedures
- Only effective prophy angle brush for orthodontic cases – use on and in between brackets, wires and bands

INSTRUCTIONS FOR USE

1. Attach the Twist prophy angle to your handpiece and run it at 4000 RPM's.
2. Maintain constant speed and contact, gliding from tooth to tooth, for as long as necessary to polish and clean all surfaces, including deep and stubborn stains. There is no need to lift, dab or dot Twist on and off the teeth. Twist will not generate any uncomfortable heat and will not stall while using firm pressure.
3. Always operate the Twist drive shaft in alignment with the axis of the handpiece. **DO NOT** flex Twist at an angle to the handpiece.

Ordering Information:

100/Bag

TPASC	Soft Cup
TPAFC	Firm Cup
TPATB	Tapered Brush
TPAFB	Flat Brush



Hauppauge, NY 11788-4209 USA

Tel: 631-582-6777 • 888-CROSSTEX • 877-PROPHYS

Fax: 631-582-1726 • E-mail: crosstex@crosstex.com



California • Georgia • New York • Pennsylvania • Argentina • Japan • The Netherlands

www.crosstex.com