

STEAM

Unprocessed



Passed Examples



Failed Examples



Special Note:

Color of the bar is not an indication of a pass or fail result. If the color bar has not entered into the blue SAFE area, DO NOT use. Proper sterilization has NOT been achieved and the items in the package should be reprocessed.

EO GAS

Unprocessed



Passed Example




Special Note:


Accept only when indicator turns Orange in EO gas. Other color variations may be indicative of suspect sterilization conditions.

VH₂O₂


Unprocessed

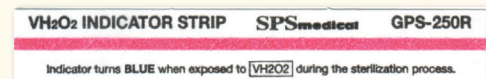

Passed Example

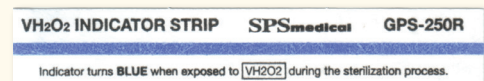

Failed Example

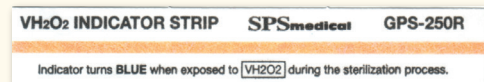

Special Note:

Accept only when indicator turns Blue when processed. Other color variations may be indicative of suspect sterilization conditions.