

# SteamPlus™ Type 5 Integrator Reference Chart

## Unprocessed



### ✓ Pass



### ✗ Fail



## SteamPlus™ Steam Sterilization Integrator Instructions for Use:

1. For the highest level of chemical monitoring from a chemical indicator, place a SteamPlus™ Integrator in each pack.
2. Process the load according to the sterilizer manufacturer's instructions.
3. Before releasing items, observe that the dark bar has entered the blue SAFE area.

### Note:

If the dark bar has not entered into the blue SAFE area, DO NOT use. Proper sterilization has NOT been achieved and the items in the package should be reprocessed. Prior to reprocessing, be sure to check the sterilizer for proper use and function.



All company and product names are trademarks of Crosstex International, Inc., its affiliates or related companies, unless otherwise noted. Marks not registered in all jurisdictions.  
© 2020 Crosstex International, Inc. All rights reserved. MLIT01574 Rev A 0920