

Wrap-Ease™ Comfort Barriers for Digital X-Rays

Large | White **REF** BWWEL / Small | Blue **REF** BWWES

INSTRUCTIONS FOR USE

- **Large | White: Adjustable to fit all size 2 sensors**
- **Small | Blue: Adjustable to fit all size 0-1 sensors**

Photos used in demonstration are Wrap-Ease™ Comfort Barriers Small, Blue.



1
Insert sensor image-side down and remove “Peel and Wrap” liner and fold over.



2
For use with Kerr Kwik-Bite™ Film Holders and/or Hawe X-Ray Film Holders. Remove entire “Peel and Stick” liner and attach to back of the holders.



3
For use with Rinn® XCP® Film Holders: remove half of “Peel and Stick” liner and attach to the back of the holder, repeat for the opposing quadrant and discard after use.



Crosstex International, Inc., a Cantel Medical Company
6789 W. Henrietta Road, Rush, NY 14543 USA
585.359.0130 | 631.582.6777 | crosstex.com



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany



Not made with natural rubber latex.

Made in China

BWWE-IFU-EN / REV B - 04/2021

CROSSTEX is a trademark of Crosstex International, Inc., its affiliates or related companies. All other product and company names referenced are trademarks of their respective owner.

©2021 Crosstex International, Inc. All rights reserved.