

INSTRUCTIONS FOR USE

Cleaning Solutions



CrossZyme™ Concentrated Ultrasonic Cleaner

Instructions for use:

- Factory sealed for product reliability and protection. To open, place container on a flat steady surface and cut seal slowly. Do not squeeze container.
- **THIS PRODUCT IS A CONCENTRATE.** Dilute 20:1 for general use and 10:1 for heavy use.
- Fill preclean soak system with properly diluted solution until approximately 2/3 full and insert basket.
- As instruments are used, place immediately into solution so enzymatic cleaning may begin. If cassettes are used, load first, close, then place in solution.
- When finished, lift basket or cassette, rinse under running water and place in ultrasonic machine filled with properly diluted Crosszyme™ Ultrasonic Cleaner.
- Operate ultrasonic machine as usual.
- For optimum cleaning results, change solution at least once per day. Helps avoid cross contamination.
- Safe on aluminum when diluted as recommended.



Ref.#'s JEZ

Contents: Bacterial Protease, Cocamidopropyl betaine, EDTA

WARNING! FOR PROFESSIONAL USE ONLY. NOT FOR INTERNAL USE.

- Use with adequate ventilation.
- Keep container closed.
- Do not add or mix with any other solution or ingredients.




WARNING

HAZARD STATEMENTS: Causes serious eye irritation. Causes skin irritation.

PRECAUTIONARY STATEMENTS: Wear protective gloves/protective clothing/eye protection/face protection. If skin irritation or eye irritation occurs, get medical attention. **IF ON SKIN:** Wash with plenty of water. Take off contaminated clothing and wash it before reuse. **IF IN EYES:** Rinse continuously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

FIRST AID: In case of contact with eyes, flush with copious amounts of water for 15 minutes. In case of ingestion, DO NOT induce vomiting. Contact INFOTRAC for 24 emergency medical treatment at 1.800.535.5053.

STORAGE & DISPOSAL: Store at room temperature 15°C-25°C (59°F-77°F). Crosszyme™ Ultrasonic Cleaner has a two (2) year shelf life from date of manufacture. Discard unused solution according to local, state and federal regulations. DO NOT reuse container.

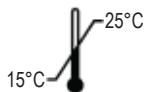

 Crosstex International, Inc.
10 Ranick Road, Hauppauge, NY 11788-4209 USA
631.582.6777 | www.crosstex.com

Made in Canada
JEZ-IFU / REV B - 09/2019

All product names are trademarks of Crosstex International, Inc., a Cantel Medical Company, its affiliates or related companies, unless otherwise noted. Marks not registered in all jurisdictions.

CROSSTEX
A CANTEL MEDICAL COMPANY