

INSTRUCTIONS FOR USE

Treatment Accessories



Flavored/Un-Flavored Dental Dams

Instructions for use:

Used to isolate and protect teeth and mouth during a dental procedure. Both latex and non-latex are available.

1. Examine mouth for which teeth need to be free from the dental dam.
2. Mark spaces on the dental dam where those teeth will be.
3. Use dental puncher to punch out those areas.
4. Fit the dental dam onto frame.
5. Fit clamp around tooth to keep the dam from moving out of place.
6. Place dam around teeth and in mouth correctly.



Made with medium modulus natural latex.



Not made with natural rubber latex.

Ref.#'s 19202, 19200, 19201, 19402,
19400, 19401, 19102, 19100,
19101, 19302, 19300, 19301, 19500



Do not reuse.



CROSSTEX[®]
A CANTEL MEDICAL COMPANY



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany



Made in Malaysia

Manufactured for: Crosstex International, Inc.
10 Ranick Road | Hauppauge, NY 11788 | 631.582.6777 | 888.276.7783 | Crosstex@crosstex.com

DDAM-IFU
Rev. 0 - 11/16