

18.5" W

25"

25"

NO PRINT AREA

WARNINGS

DANGER! Keep Out Of Reach Of Children.
Direct contact is corrosive to exposed tissue, causing eye damage and skin irritation/damage. Harmful if swallowed or inhaled. Do not get into eyes, on skin or on clothing. Avoid breathing vapor. Use with adequate ventilation. See the package insert and the Safety Data Sheet for additional information.

WARNING: This product can expose you to b-Myrcene, which is known to the State of California to cause cancer. For more information go to www.P65Warnings.ca.gov.

PRECAUTIONS

Wear goggles or face shield and chemical resistant gloves (butyl or nitrile) and liquid proof body clothing when handling or pouring. Use in well ventilated area in closed containers. Avoid contamination of food. See the package insert and the Safety Data Sheet for additional information.

STATEMENT OF PRACTICAL TREATMENT

In case of contact, immediately flush eyes or skin with plenty of water for at least 15 minutes. For eyes, get medical attention immediately. If swallowed, give large quantities of water and call a physician immediately. If inhaled, move to fresh air.

NOTE TO PHYSICIAN: Probable mucosal damage may contraindicate the use of gastric lavage. See the package insert and Safety Data Sheet for additional information.

STORAGE CONDITIONS AND EXPIRATION DATE

The expiration date for Sani Glut™ Solution is shown on the container. Store the unused and reusable solution at controlled room temperature (15°-25° C) away from sunlight. See the package insert and Safety Data Sheet for additional information.

DISPOSAL

Triple rinse empty containers with water before disposal. Do not reuse empty containers. Ensure compliance with all applicable federal, state and local regulations regarding disposal of used and unused solution.

ACTIVATION DATE: _____

EXPIRATION DATE

FOR ACTIVATED SOLUTION: _____

FOR EMERGENCY MEDICAL & SAFETY INFORMATION CALL (24 hrs) 1-800-535-5053.

Manufactured for: Crosstex International, Inc.
6789 W. Henrietta Road, Rush, NY 14543 USA
585.359.0130 | 631.582.6777 | crosstex.com

Made in USA
JGLUT / REV D - 05/2022
All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, its affiliates or related companies, unless otherwise noted.
©2022 Hu-Friedy Mfg. Co., LLC. All rights reserved.

REF JGLUT



Glutaraldehyde 3%

**28 Day Reuse
25 Minute High Level Disinfection
10 Hour Sterilization**

Reusable Sterilizing and High Level Disinfecting Solution

Glutaraldehyde	3.0%
Other Ingredients	97.0%
Total	100.0%

Keep Out Of Reach Of Children
DANGER: See side panel for additional precautionary statements

Contents: 1 Gallon (128 FL. OZ./3.8 L)



INTENDED USE

Sani Glut™ Solution is a liquid chemical sterilant and high level disinfectant intended to sterilize or disinfect reusable medical devices which contact normally sterile areas of the body and mucous membranes, respectively, when used according to the Directions for Use.

Sani Glut Solution can be reused for 28 days provided the required conditions for use (glutaraldehyde concentration, time and temperature) exist based on monitoring with a chemical indicator, timer and thermometer. **DO NOT RELY SOLELY ON DAYS IN USE.** Use patterns may reduce the established reuse life of the liquid chemical sterilant.

DIRECTIONS FOR USE

Sani Glut™ Solution is effective as a sterilant and high level disinfectant only if the devices are thoroughly cleaned, the solution is at or above the minimum effective concentration of 1.8% glutaraldehyde and the solution is used according to the Directions for Use.

PREPARATION OF SOLUTION

Activation: Activate Sani Glut™ Solution by adding the entire contents of the activator vial which is attached to the Sani Glut Solution container. Shake well. When activated, solution changes color to green. Record the date of activation on the indicated space, in a log book or on a label affixed to any secondary container used for the activated solution. See the package insert for additional instructions and information regarding the activated solution.

Sani Glut Solution is intended for use in manual bucket and tray systems made from polypropylene, ABS, polyethylene, glass filled polypropylene or specially molded polycarbonate plastics.

Sani Glut Solution **SHOULD NOT BE DILUTED.**

PREPARATION OF DEVICES

Thoroughly clean devices to be disinfected according to device manufacturer's instructions. Blood and other body fluids must be thoroughly cleaned from surfaces, lumens and objects before application of Sani Glut™ Solution. See the package insert for additional instructions and information regarding preparation of devices.

STERILIZATION

Immerse device in Sani Glut™ Solution and fill any device channels with solution, being careful to remove all air from any device channel. The device should remain in contact with Sani Glut Solution at 25° C for at least 10 hours. Remove device from the solution using aseptic technique, drain the channels and rinse according to the following directions.

Sterile Water Rinsing: Following immersion in Sani Glut™ Solution, thoroughly rinse the device by immersing in three separate copious volumes of sterile water for a minimum of one minute while flushing all channels. Use fresh portions of water for each rinse. See the device manufacturer's instructions for additional information about rinsing the device.

See the package insert for additional instructions and information regarding sterilization with Sani Glut™ Solution.

HIGH LEVEL DISINFECTION

Immerse device in Sani Glut™ Solution and fill any device channels with solution, being careful to remove all air from any device channel. The device should remain in contact with Sani Glut Solution at 25° C for at least 25 minutes. Remove device from the solution, drain the channels and rinse according to the following directions.

Rinsing: Following immersion in Sani Glut™ Solution, thoroughly rinse the device by immersing in three separate copious volumes of water for a minimum of one minute while flushing all channels. Use fresh portions of water for each rinse. See the device manufacturer's instructions for additional information about rinsing the device.

See the package insert for additional instructions and information regarding High Level Disinfection with Sani Glut™ Solution.

REUSE

Sani Glut™ Solution may be reused for a maximum of 28 days provided the minimum effective concentration of 1.8% glutaraldehyde, time and temperature are maintained. Micro-MEC™ 1.8% Glutaraldehyde Monitor Strip* should be used before each use of Sani Glut Solution to determine the glutaraldehyde concentration is above the minimum effective concentration of 1.8%.

DO NOT USE BEYOND 28 DAYS EVEN IF THE CONCENTRATION OF GLUTARALDEHYDE IS ABOVE THE MINIMUM EFFECTIVE CONCENTRATION (1.8%) AS INDICATED BY THE Micro-MEC™ 1.8% Glutaraldehyde Monitor Strip*.

See the package insert for additional instructions and information regarding the reuse of Sani Glut™ Solution.

* Micro-MEC™ is a trademark of Micro-Scientific, LLC. The Micro-MEC™ 1.8% Glutaraldehyde Monitor Strip has been verified for use with Sani Glut Solution.

CONTRAINDICATIONS

Sterilization: Routine biological monitoring is not feasible with Sani Glut™ Solution and therefore, Sani Glut Solution should not be used to sterilize reusable medical devices that are compatible with other available methods of sterilization that can be biologically monitored, e.g.: heat, ethylene oxide or peroxide gas plasma.

Sani Glut™ Solution should not be used for sterilization of critical devices intended for single use (e.g.: catheters). High Level Disinfection: Sani Glut™ Solution should not be used to high level disinfect a semi-critical device when sterilization is practical.

Endoscope Use: Sani Glut™ Solution is not the method of choice for sterilization of rigid endoscopes which the device manufacturer indicates are compatible with steam sterilization.

GLUTCROSS1-83707C-0522

NO PRINT AREA

NO PRINT AREA

NO PRINT AREA

25"

5" H

25"