
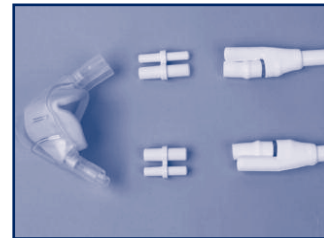




Attaching the Adaptor

- 1 On the Matrix Scavenging System, detach:
 - a) The scavenging Nasal Hood, &
 - b) The scavenging Cone.
- 2 Retain the two amber connectors in the Matrix tubes.
- 3 Attach the Accutron Parts:
 - a) The two short tubes onto Matrix amber connectors.
 - b) The two double connectors into the small Matrix tubes &
 - c) large Accutron tubes.
- 4 Attach a ClearView nasal scavenging hood onto the two double connectors.

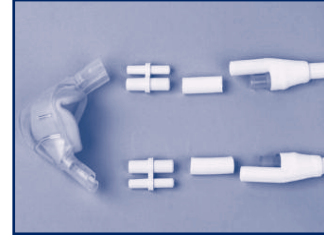

Changing the Nasal Hood for a New Patient

- 1 Remove the used ClearView nasal hood and dispose.
- 2 Clean the scavenger tubing.
- 3 Choose a new scented ClearView nasal hood per:
 - a) Proper fit to the patient
 - b) The scent the patient prefers
- 4 Attach the new ClearView nasal hood to the double connectors on the tubes.

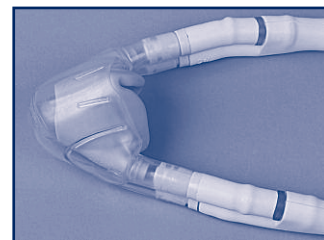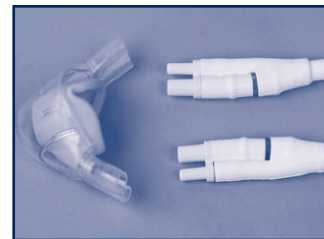

CAUTION: THE CORRECT CONNECTION OF THE “Y” COUPLING IS VERY IMPORTANT TO THE SYSTEM’S OPERATION—IMPROPER CONNECTION MAY CAUSE THE HOOD TO COLLAPSE IN USE.



1733 W. Parkside Lane • Phoenix, AZ 85027
800-531-2221 • 623-780-2020 • Fax: 623-780-0444
www.accutron-inc.com
Part No. 27734 Rev. 00



MT Promed Consulting GmbH
Allerheiligenstrasse 80
St Ingbert, D-66386 Germany
Tel: +49 6894 581 020
Fax: +49 6894 581 021