

Super

Sani-Cloth®

GERMICIDAL DISPOSABLE WIPE

TOALLITAS GERMICIDAS DESECHABLES

Use on hard, nonporous environmental surfaces.
Uselas en superficies ambientales duras y no porosas.



GENERAL GUIDELINES FOR USE INSTRUCCIONES GENERALES DE USO



1. Always dispense wipe through lid. Wear gloves when dispensing and using this product. Find center of wipe roll, twist corner of wipe into a point and thread through the hole in the canister lid. Pull through about one inch. Replace lid.

Siempre dispense limpiante por la tapa. Use guantes para suministrar y utilizar este producto. Quite la tapa, retire el sello interno y deseche. Localice el centro del rolo de toallitas; retuerza el vertice de la toallita para formar una punta y hagala pasar por el orificio de la tapa del bote. Jale aproximadamente una pulgada hacia afuera. Vuelva a colocar la tapa.



2. Cover the opening half way with one hand. Remove wipe with a uniform pull away from face and eyes. When not in use, keep lid closed to prevent moisture loss.

Cubra la mitad de la abertura con una mano. Retire la toallita de un tirón, en dirección contraria a la cara y a los ojos. Cuando no este usando las toallitas, mantenga la tapa cerrada para evitar la pérdida de la humedad.



3a. If present, remove heavy soil loads prior to disinfecting.

Si esta presente, retire las cargas pesadas del suelo antes de la desinfección.



3b. Unfold a clean wipe and thoroughly treat surface.

Despliegan un trapo limpio y completamente trate la superficie.



4. Allow surface to remain treated for two (2) minutes. Let air dry.

Permitir a la superficie tratada permanecer durante dos (2) minutos. Dejar secar al aire.



5. Do not reuse towelette. Dispose of used towelette in trash. Do not flush in toilet.

No vuelva a utilizar la toallita. Bote a la basura la toallita usada. No la tire por el inodoro.



PDI®
BE THE DIFFERENCE