


# Zap™ Disposable Dual Arch Fluoride Gel Trays

**REF** YFTHS / YFTHM / YFTHL  
INSTRUCTIONS FOR USE

You will wear your fluoride trays 5 to 10 minutes.

1. Brush and floss teeth prior to using your fluoride trays.
2. Apply a thin ribbon of fluoride gel around the inside edge of the trays.
3. Insert trays and bite together to seat the trays.
4. If excess fluoride extrudes over the edges of the trays, spit the excess out.
5. After 5 to 10 minutes, remove the trays and spit excess fluoride into the sink.
6. After your fluoride treatment, do not rinse your mouth, eat or drink anything for thirty minutes.
7. Rinse excess fluoride out of trays, dry, and store the clean trays on the mouth models so that they will retain the correct shape.

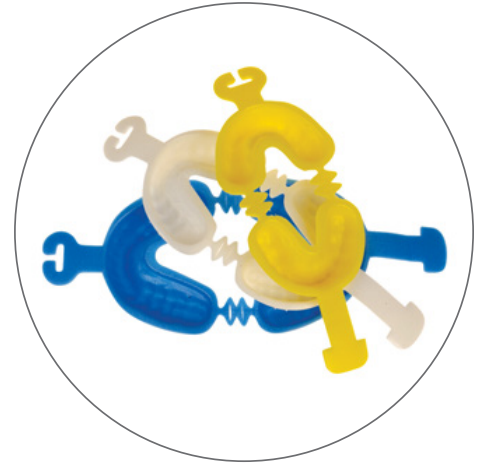

**CROSSTEX**  
A CANTEL MEDICAL COMPANY

Crosstex International, Inc., a Cantel Medical Company  
6789 W. Henrietta Road, Rush, NY 14543 USA  
585.359.0130 | 631.582.6777 | crosstex.com

EC REP

MDSS GmbH  
Schiffgraben 41  
30175 Hannover, Germany

CE

MD



Not made with natural rubber latex.

Made in China

YFTH-IFU / REV B - 05/2021

All product names are trademarks of Crosstex International, Inc., a Cantel Medical Company, its affiliates or related companies, unless otherwise noted.

©2021 Crosstex International, Inc. All rights reserved.